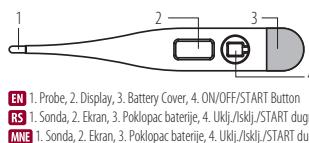
www.rossmax.com

EN 1. Probe, 2. Display, 3. Battery Cover, 4. ON/OFF/START Button

RS 1. Sonda, 2. Ekran, 3. Priklopac baterije, 4. Uklj./isklj./START dugme

MNE 1. Sonda, 2. Ekran, 3. Priklopac baterije, 4. Uklj./isklj./START dugme

EN English

Introduction

The Digital Thermometer provides quick, highly accurate reading over the body temperature range. Please read the instruction completely first. If thermometers are used by children, the supervision of the adult is compulsory. The quality of the device has been verified and conforms to the provisions of the EC council directive 93/42/EEC (Medical Device Directive) Annex I essential requirements and applied harmonized standards. EN 12470-3:2000/A1 :2009 Clinical thermometers- Part 3: Performance of compact electrical thermometers (non-predictive and predictive) with maximum device.

Attention: Consult the accompanying documents. Please read this manual carefully before use. Please be sure to keep this manual.

Battery Replacement

Place a new 1.5V D.C. button size battery type SR41 or LR41 or equivalent in the chamber with positive side faced up and negative side faced down.

Display on the LCD

M: Last measured temperature.

■: When the ■ appears on the bottom right corner of the LCD, the battery is exhausted and needs replacement.

Troubleshooting

Error message	Problem	Solution
	The system is not functioning properly	Unload the battery, wait for 1 mins and re-power it. If the message reappears, contact the retailer for service.
	Temperature taken is higher than 42.9°C (109.2°F)	Take a new temperature via close contact and sufficient rest.
	Temperature taken is lower than 32°C (90.0°F)	Take a new temperature via close contact and sufficient rest.
	Dead battery: Battery icon appears, can't be measurable.	Suggest to replace the battery.

Cleaning and Disinfection

- Clean the unit by wiping it with a dry cloth and disinfect the probe with ethyl alcohol.
- Don't let the unit contact any chemical thinner.
- Do not soak the display in water.

Caution

- Do not bend, drop or twist the thermometer. It is not shockproof.
- Do not store the unit under direct sunlight, at a high temperature, in high humidity or dust.
- Do not disassemble. See BATTERY REPLACEMENT to replace battery.
- Avoid strong electromagnetic interference such as microwave ovens and cell phones.
- Keep battery away from children. Harmful if swallowed.
- Battery should not be charged or placed into extreme heat as it may explode.
- Remove battery from the thermometer when not in operation for a long time.
- If the unit is stored in an extreme environment before measurement, please make sure it has been firstly adapted in the room temperature, so that the unit can be measured normally.
- Measurement results are for reference only. Contact your physician if you have or suspect any medical problems. Do not change your medications without the advice of your physician or healthcare professional.
- This device may not meet its performance specification if stored or used outside temperature and humidity ranges specified in specifications.

- It is recommended the performance should be checked every two years.
- Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.
- High, prolonged fever requires medical attention especially for young children. Please contact your physician.

- For safe reason, during children's temperature measurement, please keep them from crying, walking, talking and any related dangerous activities.
- Usage of the probe cover may result in a 0.1°C(0.2°F) difference from actual temperature.
- To clean the probe before and after using the thermometer to ensure an accurate reading and avoid cross contamination approximately, no matter the probe cover has been installed or not.
- Do not drink hot or cold fluids, exercise, and smoke or perform other activities prior to a reading. These activities will raise or lower temperature readings when compared to your normal, average temperature.
- Please ensure the close contact between the probe and the armpit of the baby or children.
- Do not service or maintain device while in use.

Specifications

Range:	32.0°C~42.9°C (90.0°F~109.9°F)
Accuracy:	±0.1°C 35.5°C~42.0°C (±0.2°F 95.9°F~107.6°F) at standard room temperature 18°C~28°C (64.4°F~82.4°F); ±0.2°C under 35.5°C or over 42.0°C (±0.4°F under 95.9°F or over 107.6°F)
Display:	Liquid crystal display, 3 1/2 digits
Battery:	One 1.5 V DC, button size battery (SR41 or LR41)
Power consumption:	0.15 milliwatts in measurement mode
Battery life :	Approx. 1500 times operation or 1 year with 1-2 measurement per day including standby mode.
Dimension:	12.3cmx 1.8cm x 0.9cm (LxWxH)
Weight:	Approx. 10 grams including battery
Alarm:	Approx. 10 seconds sound signal when peak temperature reached
Environmental for using:	Temperature: 5.0°C~40.0°C (41.0°F~104.0°F), Humidity: 15%~95%RH; 700~1060hPa
Storage and transportation condition :	Temperature: -20.0°C~55.0°C (-4.0°F~131.0°F), Humidity: 15%~95%RH; 700~1060hPa
IP Classification	IP22, Protection against harmful ingress of water and particulate matter
Safety classification	

RS Srpski

Uvod

Digitalni termometar omogućava brzo i visoko precizno merenje telesne temperature. Molimo Vas da pre upotrebe pažljivo pročitate uputstvo.

Ukoliko su deca korisnici termometra, preporučen je nadzor od strane odraslih osoba. Ovaj termometar je u skladu sa evropskim propisima i nosi oznaku "CE 1639". Kvalitet ovog proizvoda je potvrđen i u skladu je sa prepukom direktive EC Saveta 93/42/EEC (Direktiva o medicinskim uređajima). Aneks I osnovni zahtevi i primene utvrđene standardima. EN 12470-3:2000/A1 :2009 Klinički termometri – deo 3: Performanse kompaktnih elektronskih termometara (neintuitivnih i intuitivnih) sa maksimalnim učinkom.

Pažnja: Konsultujte prateće dokumente. Molimo Vas da pre upotrebe pažljivo pročitate uputstvo. Sačuvajte ovaj priručnik.

Zamena baterije

Postavite novu 1.5V D.C. dugmasta bateriju tipa SR41 ili LR41 ili ekvivalent u komoru sa pozitivnom stranom okrenutom ka gore i negativnom stranom okrenutom na dole.

LCD ekran

M: Poslednja izmerena vrednost telesne temperature.
■: Kada se ■ pojavi u donjem desnom ugлу ekrana, baterija je istrošena i neophodna je zamjena novom baterijom.

Rješavanje problema

Poruka o grešci	Problem	Rešenje
	Sistem ne funkcioniše ispravno.	Izljekujte bateriju, pričekajte 1 minut i ponovo je napunite. Ako se poruka ponovo pojavi, obratite se producatoru radi servisa.
	Izmerena temperatura je viša od 42.9°C (109.2°F)	Izljekujte ponovo mjerene bliskim kontaktom nakon pauze.

- It is recommended the performance should be checked every two years.
- Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.
- High, prolonged fever requires medical attention especially for young children. Please contact your physician.

- For safe reason, during children's temperature measurement, please keep them from crying, walking, talking and any related dangerous activities.
- Usage of the probe cover may result in a 0.1°C(0.2°F) difference from actual temperature.
- To clean the probe before and after using the thermometer to ensure an accurate reading and avoid cross contamination approximately, no matter the probe cover has been installed or not.
- Do not drink hot or cold fluids, exercise, and smoke or perform other activities prior to a reading. These activities will raise or lower temperature readings when compared to your normal, average temperature.
- Please ensure the close contact between the probe and the armpit of the baby or children.
- Do not service or maintain device while in use.

Specifikacije

Opseg:	32.0°C~42.9°C (90.0°F~109.9°F)
Tačnost:	±0.1°C 35.5°C~42.0°C (±0.2°F 95.9°F~107.6°F) na standardnoj sobnoj temperaturi 18°C~28°C (64.4°F~82.4°F); ±0.2°C ispod 35.5°C ili preko 42.0°C ±0.4°F ispod 95.9°F ili preko 107.6°F
Ekran:	Ekran s tečnim kristalima (LCD), 3 1/2 cifre
Baterija:	Jedna 1.5 V DC, dugmasta baterija (SR41 ili LR41)
Eskploracija:	Prosečno 1500 merenja ili 1-2 merenja dnevno u toku jedne godine uključujući standby režim
Vek trajanja baterije:	12.3 cm x 1.8 cm x 0.9 cm (LxWxH)
Dimenzije:	Prosečno 10 gramma uključujući bateriju
Težina:	Zvučni signal koji u prosекu traje 10 sekundi
Uslovi upotrebe:	Temperatura: 5.0°C~40.0°C (41.0°F~104.0°F), Vlažnost: 15%~95%RH; 700~1060hPa
Uslovi skladištenja i transporta:	Temperatura: -20.0°C~55.0°C (-4.0°F~131.0°F), Vlažnost: 15%~95%RH; 700~1060hPa
IP klasifikacija:	IP22, Zaštita od stetnog prodora vode i praškastih materija.
Klasifikacija sigurnosti:	

- It is recommended the performance should be checked every two years.
- Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.
- High, prolonged fever requires medical attention especially for young children. Please contact your physician.

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EN Warranty Card

This instrument is covered by a 5 year guarantee from the date of purchase, batteries and accessories are not included. The guarantee is valid only on presentation of the guarantee card completed by the dealer confirming date of purchase or the receipt. Opening or altering the instrument invalidates the guarantee. The guarantee does not cover damage, accidents or non-compliance with the instruction manual. Please contact your local seller/dealer or www.rossmax.com.

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Rossmax Swiss GmbH,